IMPORTANT INFORMATION

The Female Condom (FC) is free of natural rubber latex.

A female condom, also known as an internal condom, is designed to be retained in the vagina during sexual intercourse to prevent pregnancy and sexually transmitted infections (STIs).

Store in a well ventilated, cool and dry place. Away from source of heat including direct sunlight. Not to exceed average temperature of 30°C.

No barrier method, including male condoms or female condoms, provide 100% protection against pregnancy and STIs.

Important things to note for the best protection with the FC:

- Before you try the FC, be sure to read the directions and learn how to use it correctly.
- Practice putting the FC in the vagina without having sex to get familiar with how it works.

Using the FC:

- The FC is inserted before sex. Most people insert the FC between 2 to 20 minutes before sex.
- FC can be used with most commercially available sexual lubricants.

Risks of using FC:

- Use the FC correctly every time you have sex. If the FC is not used correctly every time, your risk of getting a sexually transmitted infection or becoming pregnant will be higher.
- Incorrect use may result in the FC not providing protection. To avoid risk of incorrect use, read the instructions carefully prior to use.
- The following events are rare but can occur when using the FC:
 - Discomfort during insertion
 - Pain after insertion, before sex
 - Discomfort during sex
 - Burning sensation, rash or itching

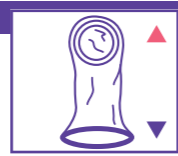
Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user lives.

Before opening:

- Check expiry date on sachet.
- Spread lubrication around by rubbing packet with hands.

Open sachet: tear down from top of sachet.

1
The FC is made of nitrile (synthetic).
▲ Inner ring, used for insertion and holds condom in place.

▼ Outer ring, covers the external genitalia.

2
Squeeze sides of inner ring together to form a point.

3
Insert squeezed ring into vagina.

4
Push up as far as it will go. Condom should not be twisted.

5
Before sex. Guide penis inside condom.

6
Before sex. Guide penis inside condom.

7
Discard in rubbish bin. Do not flush. Flushing FC would be an environmental hazard.

Hints

- Add extra lubricant if:
 - Outer ring is pushed inside the vagina.
 - Penis sticks.
 - You hear noise during sex.
- Be careful of sharp nails and jewellery to avoid tearing the condom.
- Seek medical assistance as soon as possible, at least within 72 hours, should the female condom leak or burst during use.
- If the sachet is obviously damaged discard the female condom and use a new one from an undamaged sachet.
- Consult a doctor or pharmacist about the compatibility of topical medicines that may come into contact with FC.
- FC is for single use only and cleaning and reuse can compromise the integrity of the device.

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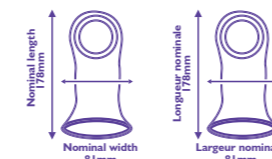
Date of manufacture
Date de fabrication

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CE 0123

The Female Health Company (UK) Plc
3 Mansfield Road
Western Avenue Business Park
London W3 0BZ UK
ISO 25841:2020

EC REP

Advena Ltd
Tower Business Centre, 2nd Flr,
Tower Street, Swatara, BKR 4013
Malta
Tel +356 2546 6689

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The Female Health Company
USA/UK/Malaysia
Issue Date: November 2023
www.femalecondom.org

G0082

Sylog Print Systems Sdn Bhd.

Model Name	G0082-Female Health fc2	Customer	The Female Health	Revision Date	30-11-2023
Open Size	A5	Colour	2 Colour	Attn to	Mr. Kumar
Barcode	-	Material	80gsm Simili	Prepared by	Zul
Code	G0082	Finishing	-	Quantity	
				Revision / Artwork No.	DC5786-4
	Customer Signature		QA UK Signature		QA Mal Signature